

AquaC UNO H

Quick Reference Guide (Acute)

3 P

P



**A
P**

**C
a**

- Verify total hardness test strips are not expired.
- Obtain a sample of RO feed water and test for total hardness.
- Top off salt level in the tank at a minimum of once each operational day.

**A
PC
T**

B

P



Q

**P
Ab**



Complete Prior to Each Additional Treatment

For additional treatments during the day, please take the following actions:

- Rinse system prior to each treatment.
- Document system operational data.
- Perform total chlorine testing in accordance with company policy.

Complete as Needed

Decalcification will be completed as the system indicates and may be required prior to system disinfection. When system indicates, complete decalcification in accordance with IFU, and document on the decalcification report prior to completing system heat disinfection.

Complete at End of Day

Avoid interrupting softener regeneration by keeping the water and drain connected for 11 minutes after switching the RO to Standby.

If applicable, complete module heat disinfection in accordance with your program protocol and IFU, and document.